

Instructions for use : • Close Roller Clamp.

- Check the package sealing integrity to ensure sterility.
- Remove the transfusion set using aseptic technique.
- Close Roller Clamp.
- Remove protective cap from closure-piercing device and insert into the outlet of blood container.
- Open the air vent cap of piercing device.
- Suspend container, half fill lower portion of drip chamber by squeezing and releasing.
- Do not fill completely.
- Remove protective cap from needle by holding needle upright, open roller clamp and allow blood to displace air entirely from the tubing and the needle.
- Perform vein puncture and regulate flow by opening roller clamp.
- This product contains latex, it may be allergic to some patients with known sensitivity to latex, It Shouldn't be used if allergic to latex.

Warning : After use, this product may be a potential biohazard. Handle and dispose off in accordance with accepted. Medical practice and applicable local, state and country laws and regulations.

Caution :

- Do not use if package is open or damaged.
- Do not use with blood or blood components.
- Do not store in direct sunlight at extreme temperature or in high humidity.
- Discard after use.
- Do not re-use.