

Instructions for use :-

- Tear the pouch and take out the burette infusion set
- Close Upper Clamp and lower Roller Clamp.
- Remove anti kink device
- Remove spike protective cap and insert spike into the fluid bottle.
- Uncap the Air filter. Open upper clamp and fill 30ml of fluid into the Burette. Close upper clamp and fill half the lower drip chamber by squeezing it. This will allow shut-off valve to float in the burette.
- Remove protective cap of I.V. Needle and open lower roller clamp.
- Remove the air bubbles in tube and close Roller Clamp.
- Open upper clamp and obtain desired volume of fluid in burette then close the upper clamp.
- Make vein puncture.
- Gradually open the lower roller clamp to adjust the drip rate and control the flow of infusion with the roller clamp.
- When more solution is required, close lower roller clamp and open upper clamp to fill burette up to desired level. Close upper clamp and squeeze the lower drip chamber a little bit to float the shut-off valve up and then restart infusion.

Cautions :

- Do not use if protective caps are detached.
- Use injection site for injecting medicine.
- Not to be used for Blood or Blood components.
- Non-Toxic

Warning : After use, this product may be a potential biohazard. Handle and dispose off in accordance with accepted medical practice and applicable local, state and country laws and regulations.