

# DISPOSABLE

## LUNG EXERCISER

### MATERIAL:

- PP, PE & ABS

### INSTRUCTIONS FOR USE:

- Hold unit in an upright position, exhale normally and then place your lips tightly around the mouthpiece.
- For a low Flow Rate, inhale at a sufficient rate to raise only the ball in the first chamber while the second chamber ball remains in the rest positions. This position should be held as long as possible. A hold manoeuvre should then be maintained for three seconds.
- For a Higher Flow rate, inhale at a rate sufficient to raise the first and second chamber balls while the third chamber ball remains in the rest position.
- Exhale. Remove mouthpiece and exhale normally, relax (repeat) following each prolonged deep breath, take a moment to rest and breathe normally. Then repeat the number of time recommended by Doctor.

### WARNING

- To be used this product under the guidance of qualified doctor or paramedic.
- Read instruction before use

### CONTRAINDICATIONS:

- do not use if patient had surgery.
- do not use if patient have lung disease

### PRECAUTIONS:

- Always take to doctor advice before starting an exercise program

## INSTRUCTIONS FOR USE

- If any changes in medicine talk to doctor before exercising
- wait for exercise at least one and half hour after eating the food.

### Adverse Reaction

- Person has develop hypersensitivity to a material or organism.

### MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCE RESULTING FROM IMPROPER USE.

- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use.
- Improper transport and handling may cause structural and / or functional damage to device or packaging.
- Do not clean or re-sterilize.
- The product should be used immediately after opening the packaging.
- The device strictly for single patient use. Multiple uses can be a potential biohazards.

### STORAGE CONDITION :

- Do not expose to heat or direct sunlight



Reference No.



Lot No./Batch No.



Manufacturer



Date Of Manufacturing



Use By / Expiry Date



Sterilized By Ethylene Oxide  
Single Sterile Barrier System



Single Use



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package Is Damaged  
Consult Instructions For Use



Caution



Consult Instructions For Use



Keep away from sunlight



Temperature Limit



Authorised Representative  
In The European Community



Medical Device



We Touch... Lives!

Disposafe Health and Life Care Limited

Plot No.1 & 2, Phase-II, Sector-59, Ballabhgarh,  
Faridabad -121004, Haryana, INDIA,  
E-mail: info@disposafe.com  
Web: www.disposafe.com  
Customer Care No. +91-129-4170201



CEpartner4U B.V., ESDOORNLAAN 13,  
3951 DB MAARN, THE NETHERLANDS,  
E-mail: office@cepartner4u.com

CE 0123

AW/IFU\_1590, Rev:01, Dt. 27-06-2023