

Thumb Scalpel

• PRODUCT

(Sterile Disposable Thumb scalpel), SS Blade fitted on plastic handle with guard.

• MATERIALS USED

Stainless Steel, ABS Plastic & PP

• INDICATIONS :

- Dissection Incision.
- Removal of extra tissue from operating area.

• CONTRAINDICATIONS:

- Re-use of blades can work as carrier for communicable disease to patient and/or user.
- Device should not be used after expiry date.

• INSTRUCTIONS FOR USE :

- Select proper type and size of Thumb scalpel as per the sight. always use bigger size for thick tissue.
- After selection clean the sight with proper disinfectant as per hospital protocol.
- Inspect package of blade if intact then remove blade from package.
- Incision has to place as per the desired length preferably measured and marked.

• CAUTION / WARNINGS:

- Re-use may cause serious health deterioration.
- The product should be used only by a qualified surgeon or paramedic.
- Before use always check integrity of product and packing along with expiry date.
- For single use only.
- Use product immediately after opening the pack

DISPOSAFE DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

INSTRUCTION FOR USE

- DISPOSAFE do not hold any responsibility if device re-used or re-sterile.
- Sterility of product is guaranteed if packaging is not damaged.
- **DISPOSAL OF USED POUCH AND BLADES:**
 - After using blade pieces and pouches shall be disposed off in compliance with the local regulatory requirements of handling medical use.
- **PRECAUTIONS:**
 - Always open the pouch from peel apart direction to avoid injury.
 - Devices are extremely sharp, use carefully while handling.
 - Care must be taken so that the pouch is not opened in an unsterile area otherwise the blade which has already been sterilized by EO Sterilization will become unsterile. Proper procedure must be used as applicable for handling any sterile product.
 - Care must be taken during of device to avoid any contact or injury due to the sharp nature of the device.
 - In case of changes in the performance of device for intended use replace the defective device by new to full fill the required application.

Disposafe Health And Life Care Limited

Plot No. 1 & 2, Phase-II, Sector-59, Ballabhgarh
Faridabad - 121004, Haryana, INDIA
E-MAIL : info@disposafe.com,
Web : www.disposafe.com
CUSTOMER CARE NO. +91-129-4170201
AW/IFU_1336_Rev_01 Date 28-09-2021

	Reference No.
	Caution/Warning (Read IFU before use)
	Do Not Reuse
	Lot Number / Batch Number
	Date of Manufacturing
	Use By / Expiry Date
	Sterilize using ethylene oxide
	Manufactured by
	Do not re-sterilize
	Keep dry
	Keep away from sunlight
	Moisture limitation 30% RH 65% RH
	Temperature Limitation 10°C 40°C
	Do not use if package is damaged
	Consult for IFU
	Non-Pyrogenic

CE **REP**

CE **0123**
CEpartner4U B.V., ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS.,
E-mail: office@cepartner4u.com