

Instructions for use :

- Close the roller clamp.
- Remove the protective cover and insert the piercing device straight into the top of the bottle up to its full length.
- Turn the bottle upside down and hang it at suitable height.
- Squeeze & release the drip chamber until it is half filled.
- Remove needle protector. holding needle upright.
- Open roller clamp and allow solution to pass until all air bubbles in the tube are removed.
- Close the roller clamp.
- Perform venipuncture and adjust the roller clamp to achieve desired flow rate.
- After use this product may be a potential biohazard and must be disposed off as per state or country laws.
- This product contains latex, it may be allergic to some patients with known sensitivity to latex, It Shouldn't be used if allergic to latex.

Caution :

- Do not use if package is open or damaged.
- Do not store in direct sunlight at extreme temperature or in high humidity.
- Discard after use.
- Do not use with blood or blood components.
- Do not re-use.

GRAVITY FEED ONLY DEHP FREE