

Disposafe

INSTRUCTION FOR USE

STERILE DISPOSABLE SCALPEL Surgical Blade Fitted On A Plastic Handle With Guard

PRODUCT

(Sterile Disposable scalpel) SS Blade fitted on plastic handle with guard.

MATERIALS USED:

- Stainless Steel, ABS Plastic & PP

INDICATIONS:

- Dissection Incision.
- Removal of extra tissue from operating area.

CONTRAINDICATIONS:

- Re-use of blades can work as carrier for communicable disease to patient and/or user.
- Device should not be used after expiry date.

INSTRUCTIONS FOR USE :

- Select proper type and size of Disposable scalpel as per the sight. always use bigger size for thick tissue.
- After selection clean the sight with proper disinfectant as per hospital protocol.
- Inspect package of blade if intact then remove blade from package.
- Incision has to place as per the desired length preferably measured and marked.

CAUTION / WARNINGS:

- Re-use may cause serious health deterioration.
- The product should be used only by a qualified surgeon or paramedic.
- Before use always check integrity of product and packing along with expiry date.

For single use only.

- Use product immediately after opening the pack
- **DISPOSAFE DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.**
- DISPOSAFE do not hold any responsibility if device

- re-used or re-sterile.
- Sterility of product is guaranteed if packaging is not damaged.

STORAGE CONDITION:

- Keep away from direct sun light.
- Keep away from rain
- Storage temperature should be 10 to 45 degree celcius.
- Humidity of storage area should be RH 35% to 65%.

DISPOSAL OF USED POUCH AND BLADES:

- After using blade pieces and pouches shall be disposed off in compliance with the local regulatory requirements of handling medical use.

PRECAUTIONS:

- Always open the pouch from peel apart direction to avoid injury.
- Devices are extremely sharp, use carefully while handling.
- Care must be taken so that the pouch is not opened in an unsterile area otherwise the blade which has already been sterilized by EO Sterilization will become unsterile. Proper procedure must be used as applicable for handling any sterile product.
- Care must be taken during of device to avoid any contact or injury due to the sharp nature of the device.
- In case of changes in the performance of device for intended use replace the defective device by new to full fill the required application.

CE 0123

CE|REP|CEpartner4U B.V., ESDOORNLAAN 13,
3951 DB MAARN, THE NETHERLANDS.
E-mail: office@cepartner4u.com

REF

Reference No



Caution/Warning (Read IFU before use)



Do Not Reuse

LOT

Lot Number / Batch Number



Date of Manufacturing



Use By / Expiry Date



Sterilize using ethylene oxide
Single Sterile Barrier System



Manufactured by



Do not resterilize



Keep dry



Keep away from sunlight



Moisture limitation



Temperature Limit



Do not use if package is damaged



Consult for IFU



Non-Pyrogenic

MD

Medical Device



Not For General Waste

DISPOSAFE HEALTH AND LIFE CARE LIMITED

Plot No. 1 & 2, Phase-ii, Sector-59, Ballabhgarh,
Faridabad - 121004, Haryana, India
E-mail : info@disposafe.com
Web : www.disposafe.com
Customer Care No. +91-129-4170201
Mfg.Lic.No.MFG/MD/2019/000106