

## **Instructions for use :**

- Close the roller clamp.
- Remove the protective cover and insert the piercing device straight into the top of the bottle up to its full length.
- Turn the bottle upside down and hang it at suitable height.
- Squeeze & release the drip chamber until it is half filled.
- Remove needle protector. holding needle upright.
- Open roller clamp and allow solution to pass until all air bubbles in the tube are removed.
- Close the roller clamp.
- Perform veinpuncture and adjust the roller clamp to achieve desired flow rate.
- After use this product may be a potential biohazard and must be disposed off as per state or country laws.
- This product should not be used on children, pregnant and nursing woman and other patient at risk.
- This product contains latex, it may be allergic to some patients with known sensitivity to it