

# INSTRUCTIONS FOR USE

## DISPOWAY I.V.CANNULA WITHOUT WINGS AND WITHOUT INJECTION PORT

### MATERIALS USED :

- PTFE/PUR/PEP,PP,HDPE,POM & Stainless Steel.

### INDICATIONS :

- Infusion of I.V. Solutions.
- Intermittent intravenous Drugs administration.
- To maintain hydration and / or correct dehydration if patient is unable to take sufficient volume of oral fluids.

### CONTRAINDICATIONS :

- Product should not be used in patient with known hypersensitivity to any of the materials used.
- Administration of highly viscous fluids.
- Large blood transfusion.

### INSTRUCTIONS FOR USE :

- Carefully select and aseptically prepare the site.
- Select suitable size of I.V. Catheter & inspect visually to as certain that package is intact.
- Remove cannula from sterile packing and remove its needle cover.
- Grip the cannula between catheter body & needle hub.
- Perform veinpuncture & check for flash back blood in flash back chamber
- Advance the catheter into vein and simultaneously withdraw the needle.

### NEVER TRY TO REINSERT THE PARTIALLY OR COMPLETELY WITHDRAWN NEEDLE.

- Withdraw the needle completely while pressing the vein just after tip of catheter into the vein & discard the needle in a appropriate container.
- Connect to the I.V. Infusion set line.
- Cover the puncture site with sterile dressing.
- Perform routine monitoring & veinpuncture site maintenance according to medical norms.

### NEVER TRY TO REINSERT THE PARTIALLY OR COMPLETELY WITHDRAWN NEEDLE.

- Withdraw the needle completely while pressing the vein just after the tip of catheter into the vein & discard the needle in an appropriate container.
- Connect to the I.V. infusion set line.
- Cover the puncture site with sterile dressing.
- Perform routine monitoring & veinpuncture site maintenance according to medical norms.

### WARNING :

- The use of this product is restricted to a qualified doctor or a Paramedic
- Read Instruction before use.
- Do not reinsert completely or partially withdrawn needle, it may lead to cut in the catheter.
- Do not press catheter while pressing the vein during catheter removal.

### MANUFACTURER DISCLAIMS ANY RESPONSIBILITY FOR POSSIBLE CONSEQUENCES RESULTING FROM IMPROPER USE.

- The product should not be reprocessed.
- Visually inspect and carefully check the product and packaging before use. Improper transport and handling may cause structural and/or functional damage to device or packaging.
- The product is guaranteed sterile & Non-pyrogenic if the package has not been opened or damaged.
- Do not clean or resterilise. For single use only. Discard after use.
- The product should be used immediately after opening the packaging.
- The device must not be re-used after its usage, it can be a potential biohazard.

### STORAGE CONDITION :

- Do not expose to heat or direct sunlight.

REF

Ref. No.

MD

Medical Device

LOT

Lot No./Batch No.



Manufacturer



Date Of Manufacturing



Use By / Expiry Date



Sterilised By Ethylene Oxide Single Sterile Barrier System



Do Not Reuse



Do Not Resterilize



Non-Pyrogenic



Do Not Use If Package Is Damaged



Caution



Consult Instructions For Use

EC REP

Authorised Representative In The European Community

10°C 45°C

Temperatre Limit



**Disposafe Health and Life Care Limited**

Plot No. 1 & 2, Phase-II, Sector-59,  
Ballabhgarh, Faridabad-121004, Haryana, India  
E-mail : info@disposafe.com, Web : www.disposafe.com  
Customer Care No. +91-129-4170201

CE REP

**C&Epartner4U B.V**

ESDOORNLAAN 13,  
3951 DB MAARN, THE NETHERLANDS.  
E-mail: office@cepartner4u.com

CE  
0123

